Easy-Up Serie 530

Brugervejledning/ User manual

Indhold/ Content

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Navn: Edith Refsgaard Gosvig, CEO

Underskrift: *Edith R. Gosvig*

Congratulations on your new Easy-Up, the helping hand!

Unpacking.

Check that all parts are included in the box. Easy-Cover and Easy-Egg are optional extra accessories for the set.

	Easy-Up ER-530	Your Easy-Up vacuum holder.
	Easy-Warm EW-410	Set with 1 silicone ring for hot pots, pans, etc. (max. 200°C).
	Easy-Mini EM-410	Set with 3 rings to hold items like pill bottles and butter/sardine cans.
	Easy-Drink ED-410	Set with 2 cups to hold plastic bottles 500ml, 1L (Tetra Pak), 300ml, and 1/2L bottles.
	USB-C cable UC-410	For charging your Easy-Up via a standard 5V USB adapter.
	Easy-Release ER-410	Tool for easily removing the vacuum from jar lids.
	Easy-Cover EC-410	Set with 3 protective covers to keep Easy-Up clean.
	Easy-Egg EE-410	Egg holder so you can peel a boiled egg.

Overview of your Easy-Up.

- 1) Charging socket with LED light
- 2) On/Off button
- 3) Hole for liquid drainage
- 4) Vacuum suction

Markings on Easy-Up.

How Easy-Up works

- Easy-Up uses vacuum to hold an item firmly to the table surface, allowing you to perform various tasks (stirring, whipping, tilting, turning, etc.) without the item moving.
- Easy-Up can be used on items like cans, jars, bowls, pots, bottles, and similar containers in the household.

Before using Easy-Up for the first time

- Charge Easy-Up. If your Easy-Up needs charging via an outlet, connect the charging cable to a standard 5V USB adapter, e.g., from your mobile phone.
- The LED light shows RED while charging and GREEN when Easy-Up is sufficiently charged.

Important information about using your Easy-Up

- Easy-Up must be placed on a horizontal surface that is completely dry, flat, and smooth.
- The item to be held must have a smooth and tight-fitting underside to create a vacuum.
- A vacuum can only be created by removing air from a cavity of a certain size. If the underside of the item is completely flat, use one of the included accessories to create the necessary cavity. Choose the accessory that best fits your item.
- Easy-Up is designed for use in private households with an expected operating time of under 2 hours per day.
- Easy-Up is not made from materials intended for direct contact with food. Food that comes into contact with Easy-Up should be discarded.

Using Easy-Up.

1. Place Easy-Up on the table.
2. Turn on Easy-Up using the toggle switch (0 → 1). The LED light turns BLUE.
3. Lace your item on top of Easy-Up, possibly on a suitable accessory.
4. Press the item lightly against Easy-Up to ensure sufficient vacuum.
5. Check if the item is held securely to Easy-Up.
6. You can now perform the desired tasks.
7. When finished, turn off Easy-Up and remove the item.

Recharging your Easy-Up

The LED light on your Easy-Up lights up RED when it needs recharging.

Cleaning your Easy-Up

- Clean Easy-Up by wiping with a damp, wrung-out cloth.
- Do not clean Easy-Up while charging.
- Do not use running water, dishwasher, or similar for cleaning Easy-Up.

Technical specifications

Weight:	0,4 kg
Dimensions:.....	Ø165 x 42 mm
Ambient Temperature:	10 – 40 °
Noise level:	<42dB
Vacuum:.....	-50 kPa
Battery	NIMH – 3xAAA, 3,6V 600 mA

Troubleshooting.

If your Easy-Up does not function as expected, you may find assistance in the table below.

Error	Cause	Solution
Easy-Up does not hold	Item does not hold firmly	Ensure Easy-Up is flat and not drawing in external air (e.g., marble or wood table).
Easy-Up does not hold 2	Insufficient vacuum	Press the item lightly against Easy-Up to ensure sufficient vacuum in the cavity under the item.
Object does not create vacuum	Insufficient cavity under the item	Use Easy Mini, Easy-Drink, or Easy-Warm to create a vacuum under the item.
Fluids in the suction hole	Fluids will come out of the little drain next to the charging port	Pour in a small amount of water to the suction hole to flush out residual fluids. DO NOT HOLD UNDER RUNNING WATER!

Safe disposal of your Easy-Up.

Easy-Up contains electronics and batteries and must be disposed of via approved collection sites (e.g., recycling centers) that handle electronic equipment. Your municipality's website can provide the location of collection sites near you. Thank you for caring for the environment!

The packaging for your Easy-Up can be disposed of as cardboard waste, but it is recommended to keep the packaging for storing your Easy-Up. This keeps all accessories together.

Warranty:

Handigood ApS provides a warranty for material and production defects under normal usage scenarios. The warranty is valid for 36 months from the date of purchase or delivery. The warranty may be void if the product is used contrary to intended usage, modified, or attempted to be repaired by unauthorized personnel. The warranty covers only the replacement or repair of the product, and any consequential damages resulting from a product defect are not covered. For more information, visit: <https://handigood.com/terms-and-conditions/>

Certification:

CE Declaration of Conformity

We,

Handigood ApS, Lilleringvej 14B, 8462 Harlev, Denmark

Hereby declare that

Product: Easy-Up, Model ER-530

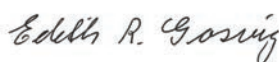
Produced from: May, 2024

Is manufactured in accordance with

- The Electromagnetic Compatibility Directive (EMC): 2014/30/EU
- The European ROHS Directive (RoHS): 2002/95/EU

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